

#### 4. Activation

Push plunger rod to activate the safety mechanism. Depress until you sense two "clicks".

*Activation can occur before or after the needle is withdrawn from the patient.*



#### 5. Needle Retraction

The needle will disappear into the syringe and the plunger will be recessed into the barrel when activation is complete.

*Discard immediately in the nearest sharps container.*



#### Don't Forget...

Aseptic technique and proper skin preparation of the site are essential. Follow your institution's policy for safe disposal of all medical waste.

The BD Integra™ Syringe is available in an array of sizes for all SubQ and IM injections.

BD Integra™ 3mL Syringe with Detachable Needle			
Catalog #	Description	Gauge/Needle	Packaging
305269	3mL	25G x 5/8"	100/Box – 400/Case
305270	3mL	25G x 1"	100/Box – 400/Case
305271	3mL	23G x 1"	100/Box – 400/Case
305272	3mL	22G x 1½"	100/Box – 400/Case
305273	3mL	21G x 1"	100/Box – 400/Case
305274	3mL	21G x 1½"	100/Box – 400/Case
BD Integra™ Needles			
Catalog #	Description	Gauge/Needle	Packaging
305310	Retracting Ndl.	25G x 5/8"	100/Box – 1000/Case
305311	Retracting Ndl.	25G x 1"	100/Box – 1000/Case
305312	Retracting Ndl.	23G x 1"	100/Box – 1000/Case
305313	Retracting Ndl.	22G x 1½"	100/Box – 1000/Case
305275*	Blunt Fill Ndl.	18G x 1½"	100/Box – 1000/Case
305833*	Blunt Filter Ndl.	18G x 1½"	100/Box – 1000/Case
3mL Syringe Only			
Catalog #	Description	Gauge/Needle	Packaging
305283	3mL Syringe Only		100/Box – 400/Case

\*18G Blunt Fill and Filter Needles have a non-retracting needle

#### REMEMBER:

Safety-Engineered Devices are required in your Custom and Standard Procedure Trays.

For more information call  
**1.888.237.2762**



BD Medical  
1 Becton Drive  
Franklin Lakes, NJ 07417  
www.bd.com

BD, BD Logo, and all other trademarks are property of Becton, Dickinson and Company. © 2013 BD.  
BD 1/13-MSS0072



Helping all people live healthy lives

**BD Integra™ Syringe**  
with *Retracting*  
**BD PrecisionGlide™ Needle**

## How to Use the BD Integra™ Syringe

### 1. Attachment

Attach the BD Integra™ needle onto the BD Integra™ syringe with a push and a clockwise twist. If pre-attached, tighten the needle to the syringe prior to use.



### 2. Aspiration

Pull the shield directly off of the needle. Draw up medication in accordance with established protocol.




### 3. Injection

Administer medication in accordance with established protocol, making sure that all medication has been dispensed.



## Tru-Lok™ Technology helps to lower medication waste

The BD Integra™ Syringe, with Tru-Lok™ technology, is designed to have significantly lower waste space volume as compared to a traditional BD luer connection.\* Less waste space can reduce drug wastage.

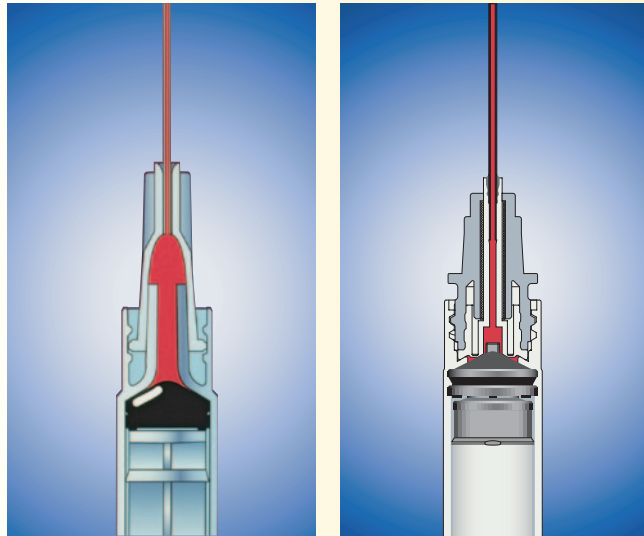

BD™ Conventional  
3ml Syringe

BD Integra™  
3ml Syringe

\* Data on file at BD


## The BD Integra™ Syringe combines innovative technology with the BD quality and comfort you trust.

### Prior to Activation

#### Single-Handed Activation

#### Tru-Lok™ Technology

Ensures low waste space and a reliable connection

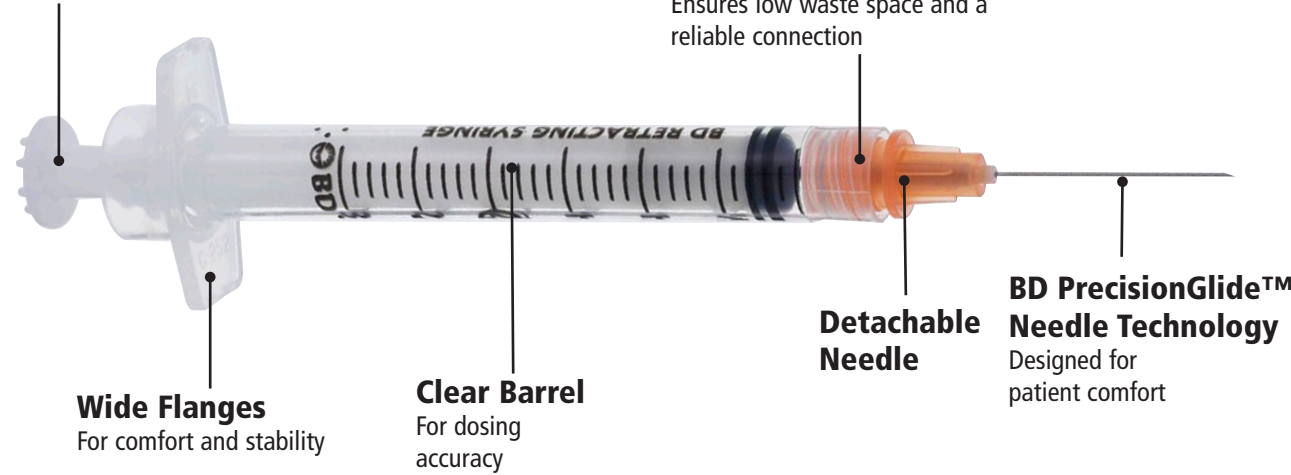

### After Activation



## The BD Integra™ Syringe Advantage



#### BD PrecisionGlide™ Needle Technology

- Designed for patient comfort
- Leading brand of hypodermic needle technology in the U.S.

#### Tru-Lok™ Technology

- Provides low waste space
- Creates a secure connection

#### Detachable Needle

- Allows clinicians to change needles for aspiration and administration

#### Safe, One-Handed Activation

- Retracting needle disappears into the syringe barrel
- Eliminates exposure to used needle
- Requires virtually no change in technique

#### Activation Choice

- Activates before or after needle is withdrawn from the patient
- Complete dosing is ensured prior to activation