Welch Allyn Two Piece Blood Pressure Cuff

Mat'l No. 703041 Rev. B

Warnings

WARNING Intravenous Systems (IV) - Do not connect cuffs with luer lock connectors to intravenous fluid systems or air may enter patient.

WARNING Size - Only use the cuff when the artery index marker falls within the printed range indicated on the cuff, otherwise erroneous readings may result.

WARNING Legibility - If artery index or range markings are not visible, do not use the cuff.

WARNING Do not allow the cuff to remain on a patient for more than 10 minutes when inflated above 10 mmHg. This may cause patient distress, disturb blood circulation, and/or contribute to the injury of peripheral nerves.

Cleaning:

Before washing the cuff, remove the inflation bulb (if available) and bladder (see diagram below). Place the hook and loop fasteners in the closed position.

Durable Cuffs

Clean with a damp cloth or wash in water with soap or detergent. Allow cuff to air dry.

Caution DO NOT PRESS WITH AN IRON.

Warranty

Durable Two-Piece Cuffs: Two-year warranty against original defects in material or workmanship. This warranty does not cover breakage or failure due to tampering, misuse, neglect, accidents, modification, or shipping. The warranty is void if the blood pressure cuff is not used in accordance with manufacturer's recommendations or if repaired or serviced by other than Welch Allyn or a Welch Allyn authorized representative. Send all repairs prepaid to:


Welch Allyn, Inc. Repair Department
4341 State Street Road
Skaneateles Falls, NY 13153, USA
Telephone: 800-535-6663 (USA only) or 315-685-4560

This product conforms to applicable sections of European Standards:

EN 1060-1, Non-invasive sphygmomanometers, Part 1.
General Requirements and EN 1060-2, Part 2. Supplementary requirements for mechanical sphygmomanometers.

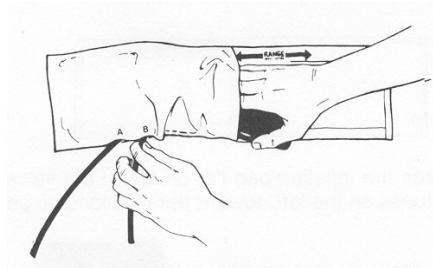The CE mark on this product indicates that it is in conformity with the provisions of Council Directive 93/42/EEC.

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