

## Walking Aid

### Aluminum Crutches

#### Characteristics

- Made of lightweight, corrosion-resistant ionized aluminum
- Height adjustable handgrips
- Push-button enables quick and safe height adjustment, by intervals of 2.5 cm (1")
- Height recommendations identified on the centre tube
- Equipped with the following accessories:
  - 934 - Tips
  - 936 - Cushions
  - 938 – Handgrips

#### Available models

CODE	USER HEIGHT	SIZE
4031	1.38 – 1.58 m (4'6" – 5'2")	SMALL
4032	1.58 – 1.79 m (5'2" – 5'10")	MEDIUM
4033	1.79 – 1.99 m (5'10" – 6'6")	LARGE

#### How to adjust crutches

1. Keep the body straight, shoulders relaxed.
2. The top of the crutch should be 2 " (5 cm) below the armpit.
3. Hands restraints are aligned with the wrists.
4. The crutch shoe must be at 6 " (15 cm) from the foot.

