420216A

# UrgoTul Ag/Silver

FR

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# UrgoTul Ag/Silver

EN

## LIPODO-COLLOID CONTACT LAYER IMPREGNATED WITH SILVER SALTS

### DESCRIPTION

**UrgoTul Ag/Silver** is a dressing impregnated with Silver salts produced using Lipido-Colloid Technology (TLC), an innovation developed by Laboratoires URGO.

The dressing consists of a Silver Healing Matrix made of polyester mesh impregnated with hydrocolloid particles (carboxymethylcellulose), vaseline, cohesion polymers and Silver salts.

**UrgoTul Ag/Silver** is non-adhesive and non-occlusive and does not adhere to the wound bed.

### PROPERTIES

In contact with wound exudate, the hydrocolloid particles (CMC) gel and combine with the matrix to form a lipido-colloid film.

This results in specific properties:

- Maintenance of a moist environment that promotes healing.

- Gelling in contact with the wound.

- Removal which does not damage newly formed tissue.

- Pain-free dressing changes for the patient.

- Protection of the surrounding skin.

Moreover, Ag+ ions of the Silver Healing Matrix (TLC-Ag) confer to **UrgoTul Ag/Silver** an antibacterial activity able to reduce local bacterial load.

Silver has a broad spectrum of anti-microbial activity covering the Gram-negative and Gram-positive bacteria and certain moulds and yeasts. It is particularly effective against *Staphylococcus aureus*, MRSA, *Streptococcus pyogenes* and *Pseudomonas aeruginosa* (pyocyanic bacillus) which are most frequently responsible for infected wounds. The dressing demonstrates in vitro antibacterial activity for up to 7 days. The antibacterial effect of the dressing also contributes to odour reduction.

### INDICATIONS

**UrgoTul Ag/Silver** is indicated for the local treatment of wounds at risk or with signs of local infection: chronic wounds (pressure ulcers and leg ulcers) and acute wounds (partial thickness burns, dermabrasions, traumatic wounds, surgical wounds, etc...).

Due to the non-adhesive nature of the dressing, **UrgoTul Ag/Silver** is recommended in the treatment of wounds in which the skin around the lesion is weakened.

### INSTRUCTIONS FOR USE

- Cleanse the wound with saline solution. If an antiseptic is first used, rinse the wound thoroughly with saline solution before applying **UrgoTul Ag/Silver**.

- If needed, **UrgoTul Ag/Silver** can be cut using sterile scissors to match the size of the dressing to the wound.

- Remove the protective wings from the dressing.

- Apply the **UrgoTul Ag/Silver** dressing directly onto the wound.

- Cover **UrgoTul Ag/Silver** with a secondary dressing (sterile pad suitable for the absorption of exudate, to be secured in place with a fixing bandage).

- Apply a compression bandage when prescribed.

- The **UrgoTul Ag/Silver** dressing can be changed every 1 to 3 days depending on the wound treated and the course of its healing.

- The maximal duration of the treatment with **Urgotul Ag/Silver** is one month.

### WARNINGS AND PRECAUTIONS

- Treatment with **UrgoTul Ag/Silver** should be carried out under medical supervision.

- Use of this dressing does not negate the need for appropriate systemic anti-bacterial treatment of infected wounds in line with local policy.

- **UrgoTul Ag/Silver** can adhere to surgical gloves (latex). Consequently, it is recommended that the gloves be moistened with normal saline in order to make it easier to hold the dressing.

- Concomitant use with other local treatments is not recommended.

- Do not use in pregnant or breast-feeding women, newborn and premature babies in the absence of any specific clinical data.

- Clinicians / Healthcare Professionals should be aware that there are very limited data on prolonged and repeated use of silver containing dressings, particularly in children and neonates.

- Avoid contact with electrodes or conductive gels during electronic measurements, e.g. EEG and ECG.

- Do not use in a hyperbaric chamber.

- Single-use, individual and sterile dressing: The reuse of a single use product may entail risks of infection.

- (\*) Check that the sterile protector is intact before use. Do not use if package is damaged.

- Do not re-sterilise.

### CONTRA-INDICATIONS

- Known sensitisation to Silver and other ingredients of the dressing.

- Do not use on patients undergoing Magnetic Resonance Imaging (MRI) examination.

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