




## REUSABLE LATEX LEG BAG CLEANING & MAINTENANCE INSTRUCTIONS

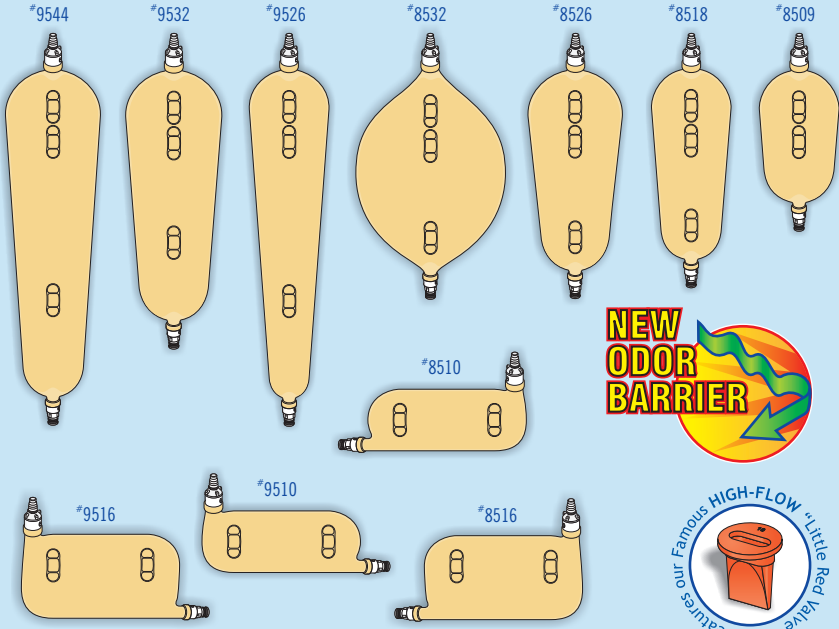

| ITEM # | DESCRIPTION        | DIMENSIONS†  |                 | CAPACITIES† |                  |
|--------|--------------------|--|-----------------|-------------|------------------|
|        |                    | Inches(″)  | Centimeters(cm) | Ounces(oz.) | Milliliters(ml.) |
| 9544   | Extra-Large        | 20 <sup>5</sup> / <sub>8</sub> x 4 <sup>3</sup> / <sub>4</sub> | 52.4 x 12.1     | 44          | 1,300            |
| 9532   | Large              | 16 <sup>3</sup> / <sub>4</sub> x 4 <sup>3</sup> / <sub>4</sub> | 42.5 x 12.1     | 32          | 1,000            |
| 9526   | Long-Slim          | 20 <sup>5</sup> / <sub>8</sub> x 3 <sup>3</sup> / <sub>4</sub> | 52.4 x 9.5      | 26          | 800              |
| 9516†  | Sport-Right, Large | 5 <sup>3</sup> / <sub>4</sub> x 10                             | 14.6 x 25.0     | 16          | 500              |
| 9510†  | Sport-Right, Small | 4 <sup>3</sup> / <sub>4</sub> x 10                             | 12.1 x 25.0     | 10          | 300              |
| 8532   | Large-Oval         | 13 x 7   | 33.0 x 17.8     | 32          | 1,000            |
| 8526   | Short-Wide         | 13 <sup>3</sup> / <sub>8</sub> x 4 <sup>3</sup> / <sub>4</sub> | 34.0 x 12.1     | 26          | 800              |
| 8518   | Medium             | 13 x 3 <sup>3</sup> / <sub>4</sub>                             | 33.0 x 9.5      | 18          | 550              |
| 8516†  | Sport-Left, Large  | 5 <sup>3</sup> / <sub>4</sub> x 10                             | 14.6 x 25.0     | 16          | 500              |
| 8510†  | Sport-Left, Small  | 4 <sup>3</sup> / <sub>4</sub> x 10                             | 12.1 x 25.0     | 10          | 300              |
| 8509   | Small              | 9 <sup>3</sup> / <sub>4</sub> x 3 <sup>3</sup> / <sub>8</sub>  | 24.8 x 9.2      | 9           | 275              |

† Approximate dimensions include adaptors and drain valves. Approximate volumes in fluid ounces and milliliters.

† These leg bags are designed to be worn horizontally above the hemlines of skirts, dresses and swimsuits.

## IMPORTANT! READ THIS FIRST:

Rinsing alone does not clean a urinary appliance. Use Urolux<sup>®</sup> urinary appliance cleanser & deodorant (#700216). Do not use boiling water, vinegar, bleaches or toilet bowl cleaners; these chemicals will damage or distort plastic and latex parts.

## BOTTOM DRAIN VALVE INSPECTION:

Twist the standard bottom drain valve (#6004) 1½ turns counter-clockwise to open, Reverse to close. TAKE CARE NOT OVER-TIGHTEN THE DRAIN VALVE WHEN CLOSING! If the valve becomes difficult to open/close, remove the valve from the valve housing completely and lubricate the black "O"-Ring with a smear of petroleum jelly—see Figure 2. Then reinsert.

## LEG BAG CLEANING PROCEDURE:

**STEP 1:** Close bottom drain valve (#6004). Unscrew the standard top adaptor (#6002) and remove the red anti-reflux valve (#6001)—see Figure 1.

**STEP 2:** Then squeeze one-half ounce (15 ml.) of Urolux<sup>®</sup> solution (#700216) through the valve housing into the bag followed by one cup (250 ml.) of warm water. Reinsert the red anti-reflux valve into its original position and replace the standard top adaptor.

**STEP 3:** Connect a rubber tube to the end of the standard bottom drain valve (#6004)—If you have product #6005 Straight-Thru Adaptor with thumb clamp, it may be substituted for the rubber tube. While holding both ends of the latex leg bag in an upright position as shown in Figure 3, open the standard bottom drain valve by twisting it 1½ turns counter-clockwise; then connect the opposite end of the rubber tube to the standard top adaptor as shown.

**STEP 4:** Agitate contents of bag periodically over a 15 minute soaking; this will allow the Urolux<sup>®</sup> solution to circulate throughout the bag reaching all urine affected areas. Regulate the strength of the Urolux<sup>®</sup> solution and cleaning time to the condition of the bag to be cleaned.

**STEP 5:** After 15 minutes has expired, return the bag to an upright position, remove the rubber tube from the top adaptor and close the bottom drain valve by twisting it 1½ turns clockwise. Dispose of used cleaning solution and flush contents of leg bag thoroughly with water. Wash outside of bag with a mild hand soap and towel dry. Dust outside of bag with talcum powder. When alternating bags, store alternate bag in a dry, cool, dark place.


**NOTE:** If urine crystals or calcium build-up remain after cleaning, use of the Small Parts Cleansing Brush (#7003) may be necessary to help dislodge them. This brush is made with durable, white-nylon bristles and a malleable, twisted, stainless-steel wire body. It may be shaped to fit almost any contour and is especially good for removing uric crystal build-ups from tight corners, small holes and other hard to reach areas within urinary and ostomy appliances—see Figure 3.



### FIGURE 1

Cut-away view of the top valve housing assembly with the red, anti-reflux valve seated in position for use.


**CAUTION:** Since our “Little Red Anti-Reflux Valve” serves two purposes; one, as an anti-reflux and two, as a Gasket-Ring (#6000) to prevent leaking, it is important that the valve remain in place during use. If the valve is left out or removed for any reason, leaks around the top valve housing may occur. To prevent this, we recommend the use of Gasket-Ring #6000. This will seal off the threads and prevent any possible leaks—this product is sold separately and is not included in the standard package.

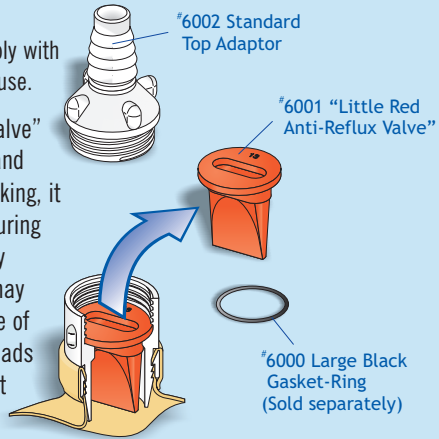

### FIGURE 2


View of bottom valve housing with standard bottom adaptor removed.

Twist clockwise to close and counter-clockwise to open. **IMPORTANT:** Do not over-tighten drain valve when closing.

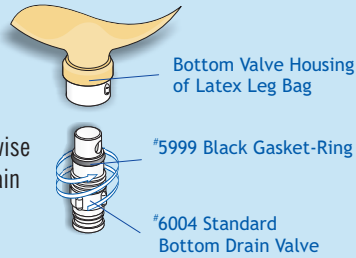

### FIGURE 3

Bag with rubber tube attached for cleaning.

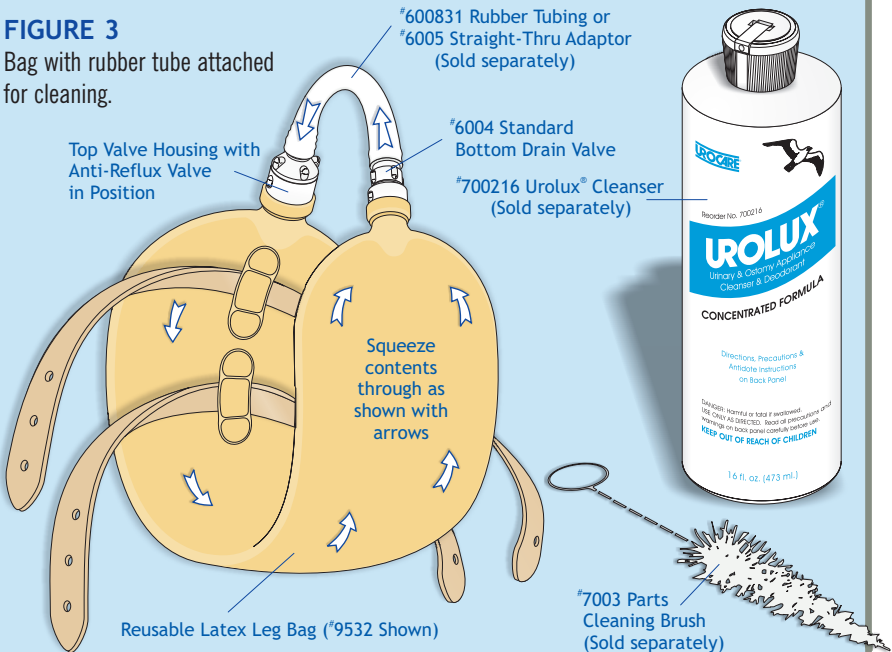



## RELATED PRODUCTS

Urocare® Reusable Latex Leg Bags

Uro-Safe® Vinyl Leg Bags

Drainage Systems and Kits

Uro-Con® & Uro-Cath® Male External Catheters

Drain Valves & Adaptors

Drainage Bottles

Uro-Prep® Protective Skin Barrier Wipes

Fabric Leg Bag Holders and Straps

Catheter Straps

Uro-Bond® Brush-on Silicone Adhesive

Urofoam® Adhesive Foam Strips

Adhesive Remover Pads

Urolux® Urinary/Ostomy Appliance Cleanser and Deodorant

Connectors and Clamps

Small Parts Cleansing Brush

Extension and Drainage Tubing

DRI Sleeper™ Bed-Wetting Treatment Alarms

For more information on any of Urocare's other products, contact your nearest Urocare dealer or write to:

Urocare Products, Inc.  
Customer Service Department  
2735 Melbourne Avenue  
Pomona, CA 91767-1931, U.S.A.  
T: (909) 621-6013 • F: (909) 621-4436  
E: [sales@urocare.com](mailto:sales@urocare.com) • I: [www.urocare.com](http://www.urocare.com)



PLEASE RECYCLE  
Printed on  
Recycled Paper

FORM NO: P19032EN-99820101D