

Références bibliographiques

- 1 Leichsenring I, Plesch W, Unkrig V, et al. Multicentre ISI assignment and calibration of the INR measuring range of a new point-of-care system designed for home monitoring of oral anticoagulation therapy. *Thromb Haemost* 2007;97:846-861.
- 2 Plesch W, Van den Besselaar AMHP: Validation of the international normalized ratio (INR) in a new point-of-care system designed for home monitoring of oral anticoagulation therapy. *Int Jnl Lab Hem* 2009;31:20-25.
- 3 Clinical and Laboratory Standards Institute: Protection of Laboratory Workers from Occupationally Acquired Infections; Approved Guideline – Third Edition; CLSI document M29-A3, 2005.
- 4 Ansell J, Jacobson A, Levy J et al. Guidelines for implementation of patient self-testing and patient self-management of oral anticoagulation. International consensus guidelines prepared by International Self-Monitoring Association for Oral Anticoagulation. *Int J Cardiol* 2005;99:37-45.
- 5 Menéndez-Jándula B, Souto J et al. Comparing Self-Management of Oral Anticoagulant Therapy with Clinic Management. *Ann Intern Med* 2005;142:1-10.
- 6 Taborski U, Wittstamm F-J, Bernardo A: Cost-effectiveness of self-managed anticoagulant therapy in Germany. *Sem Thromb Hemostas* 1999;25:103-109.
- 7 Tripodi A, Breukink-Engbers WGM, van den Besselaar AMHP: Oral Anticoagulant Monitoring by Laboratory or Near-Patient Testing: What a Clinician Should Be Aware Of. *Sem Vasc Med* 2003;3:243-254.
- 8 Moll S and Ortel TL Monitoring Warfarin Therapy in Patients with Lupus Anticoagulants. *Annals of Internal Medicine* 1997;127:177-185.
- 9 Plesch W, Wolf T, Breitenbeck N et al. Results of the performance verification of the CoaguChek XS System. *Thromb Res* 2008;123:381-389.
- 10 Deom A, Reber G, Tsakiris DA et al. Evaluation of the CoaguChek XS Plus system in a Swiss community setting. *Thromb Haemost* 2009;101:988-990.
- 11 Multicenter Performance Evaluation of the CoaguChek XS System; Evaluation Report from Roche Diagnostics available upon request.

Symboles

Roche Diagnostics utilise les signes et les symboles suivants pour ce produit.

REF

Référence du catalogue