



CIDEX®

ACTIVATED GLUTARALDEHYDE SOLUTION

HIGH-LEVEL DISINFECTION

For over 45 years, CIDEX® Activated Glutaraldehyde Solution has been used and recognized as a worldwide trusted brand for effective high-level disinfection of flexible endoscopes and other medical devices.

EXPERIENCE THE BENEFITS

CIDEX® Activated Glutaraldehyde Solution contains 2.4% glutaraldehyde and has been tested and cleared as a high-level disinfectant.

- **Effective**
- **High compatibility with materials**





ACTIVATED GLUTARALDEHYDE SOLUTION

FEATURES & BENEFITS

- **EFFECTIVE** – achieves high-level disinfection in 20 minutes at 20°C.
- **LONG-LASTING EFFICACY** – reusable for up to 14 days when monitored with CIDEX® Solution Test Strips.
- **EXCELLENT MATERIALS COMPATIBILITY** – can be used safely to disinfect a wide range of instruments, reducing risk of damage and associated cost of repairs.
- **EASY CONTROL OF EFFICACY** with CIDEX® Solution Test Strips.

MATERIALS COMPATIBILITY

CIDEX® Activated Glutaraldehyde Solution offers excellent materials compatibility and can therefore be used to disinfect a wide range of medical instruments, made of aluminum, brass, copper, stainless steel, plastics and elastomers. Refer to instructions for use for complete details.

MICROBICIDAL ACTIVITY

CIDEX® Activated Glutaraldehyde Solution provides a wide spectrum efficacy against bacteria, mycobacteria, viruses and fungi. The solution can also achieve some sporicidal activity with longer exposure time.

TECHNICAL INFORMATION

| | |
|-------------------------------|--|
| MEDICAL DEVICE CLASSIFICATION | Class IIb according to MDD 93/42/EEC |
| IN-USE CONCENTRATION | 2.4% Glutaraldehyde |
| SOAK TIME | High-level Disinfection: 20 minutes at 20°C Sporicidal activity: 10 hours at 25°C |
| USE LIFE | Up to 14 days |
| SHELF LIFE | 12 months |
| OPEN BOTTLE SHELF LIFE | 14 days (when open and activated) |
| STORAGE | 15-30°C |
| DISPOSAL | Drain or as per hospital policy. Flush thoroughly with water. |

ORDERING INFORMATION

| REORDER NO. | DESCRIPTION | CASE CONTENTS |
|-------------|--|----------------|
| SCX145 | CIDEX® Activated Glutaraldehyde Solution | 4 x 5 liters |
| MCX001 | CIDEX® Solution Test Strips | 12 x 60/bottle |
| MCX002 | CIDEX® Solution Test Strips | 2 x 15/bottle |



CIDEX® Activated Glutaraldehyde Solution may also be used for instrument reprocessing in automated equipment, in accordance with equipment manufacturers' recommendations.

Please read and follow the Instructions for Use prior to using CIDEX® Activated Glutaraldehyde Solution for detailed information, including contraindications, warnings and proper directions for use.

FOR MORE INFORMATION, CONTACT YOUR LOCAL ADVANCED STERILIZATION PRODUCTS SALES REPRESENTATIVE

www.aspjj.com/emea

| | | | | |
|--|--|--|---|--|
| <p>UK: Advanced Sterilization Products A Division of J&J Medical Ltd. Pinewood Campus Nine Mile Ride Wokingham Berkshire RG40, 3EW, England T: +44 1344 871 081 F: +44 1344 871 171</p> | <p>Ireland: Advanced Sterilization Products A Division of J&J Medical Ireland Airton Road, Tallaght, Dublin 24, Ireland T: +353 1 466 5200 F: +353 1 466 5340</p> | <p>Egypt: Johnson & Johnson Medical Egypt Florida Mall 5th Floor 1229 Square El Sheikh Ali Gad El Hak St. Heliopolis Cairo, Egypt T: +202 2268 5026 F: +202 2268 4674</p> | <p>Middle East: Johnson & Johnson Middle East FZ LLC Mohamed Bin Rashid Academic Medical Centre, Building 14, Level 7, Dubai Healthcare City, PO Box 505080, Dubai, United Arab Emirates T: +9714 4297 200 F: +9714 3314 034</p> | <p>South Africa: Advanced Sterilization Products A Division of J&J Medical Pty. Ltd. SA PO Box 273, Midrand Halfway House 1685 South Africa T: +27 11 265 1120 F: +27 11 265 1189</p> |
|--|--|--|---|--|

ADVANCED STERILIZATION PRODUCTS

Division of Cilag GmbH International
a **Johnson & Johnson** company